

COVID-19 NASAL SWAB COLLECTION KIT

INSTRUCTIONS

Failure to follow these instructions may lead to inaccurate test results.

1... Specimen Collection



1. Open the collection kit. Of the 2 swabs included, select the one with the large tip.




2. Remove swab from container, being careful not to touch the soft end.



3. Insert the entire absorbent tip of the swab into your nostril. Do not insert more than $\frac{3}{4}$ inch into your nose.



4. Slowly rotate the swab in a circular path against the inside of your nostril at least 4 times for 15 seconds.

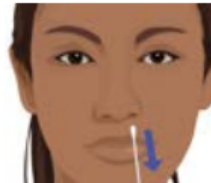

5. Gently remove the swab.



6. Using the same swab, repeat steps 3-5 in your other nostril.

2... Preparation of Sample for Return



1. Place the swab in the tube and snap off the end at the break line. Tightly seal cap on tube.



2. On label, write your name, date of birth, and collection date/time. Affix label to tube as shown after completing collection.



3. Place the tube containing the swab in the biohazard bag and seal the bag.



4. Return specimen to MAWD with Manifest paperwork.